


Description Protective Visor 275000

The visor is used to reduce the risk of drip contamination. It has been developed in response to the rapidly growing need for protective equipment and as a result of the COVID-19 pandemic. It is intended as a one-time protection, but the user decides whether or not to use it multiple times. The visor meets the applicable basic health and safety requirements in accordance with MET5659, in addition to the requirement for clarity (EN 166 §7.1.2.3 Diffusion of light). The product may not be distributed or resold on the open market. **Before using the visor, it should be cleaned as instructed for cleaning on the next page.**

Instructions

Mount the visor (2) on the arch (1) by placing the hooks on the arch in the holes on the visor. Hook the elastic band (3) on the bow.

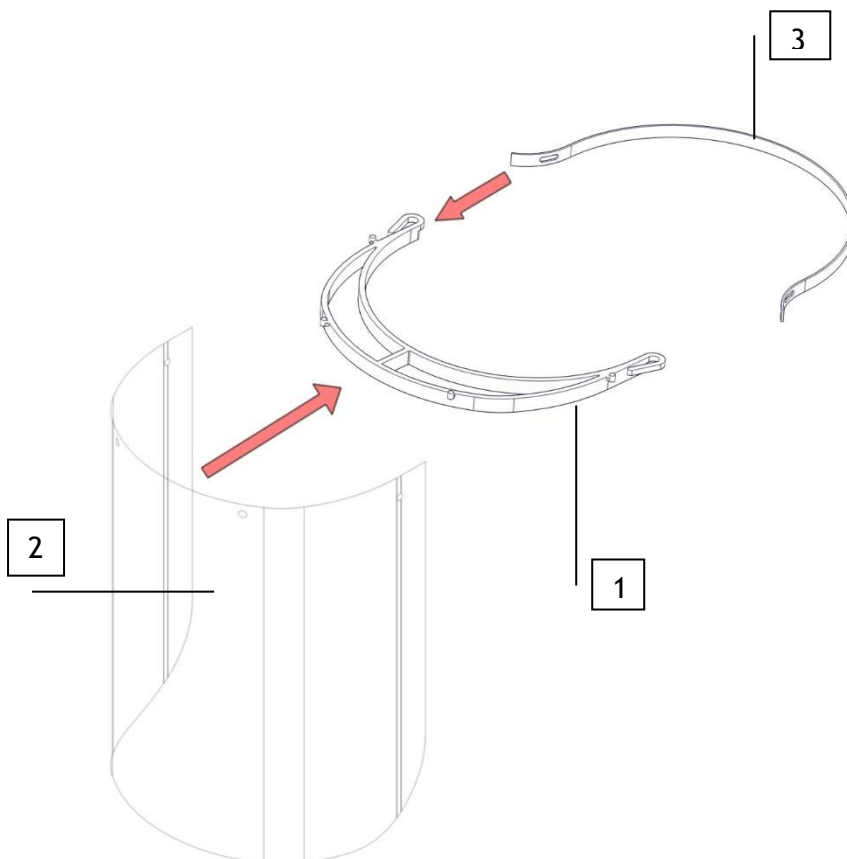

Cleaning and disinfection of visor

Reuse of visor

The visor is for single use only. Protective visors older than 3 years after manufacture should not be used. The visor can be reused as long as the functionality including fit and visibility is intact. Visas to be reused are cleaned and disinfected immediately following the removal instructions immediately after use.

Instructions for cleaning and disinfecting visors

1. Disinfect hands with hand disinfectant.
2. Wear protective gloves.
3. Clean and disinfect visors with alcohol-based surface disinfectant with cleaning effect. Use plenty of surface disinfectant and mechanically wipe with tissue paper.
4. Do not forget to disinfect the inside of the visor and remember to take new material for disinfection between the outside and the inside.
5. Hang or place the visor so it can dry.
6. Hang or place the visor so it can dry.
7. Disinfect hands with hand disinfectant.

Instruction for disinfection of multi-bow

1. Disinfect hands with hand disinfectant.
2. Wear protective gloves.
3. Remove the transparent interchangeable visor and throw in a hazardous waste box.
4. Remove protective gloves.
5. Disinfect hands with hand disinfectant.
6. Wear protective gloves.
7. The multipurpose arc is disinfected with alcohol-based surface disinfectant.
8. Hang or place the bow so it can dry.
9. Remove protective gloves.
10. Disinfect hands with hand disinfectant.

After use

Be careful not to touch the visor on the front to avoid contact with any contaminants or infectious particles. The product should be treated as hazardous waste after use.

This manual should be made available to each user. Available at www.accentaplast.se/visir

The protection visor is approved by the Swedish Work Environment Authority to 2021-06-30. Do not use afterwards.